

## SS Series Trapeze

### USER MANUAL

## SPECIFICATIONS

---

- Maximum allowable weight is 250 lbs
- Tubular construction for maximum weight to strength ratio
- Handle can be adjusted from 22" to 45" above the sleep surface
- Reach can be adjusted from 15" to 30" from the headboard
- The tube extending over the bed can be placed in 5 unique positions
- Overall maximum height is 81"

## INSTALLATION INSTRUCTIONS

---

### ***Tools Required:***

(2) 9/16 Wrench Open Ended  
9/16 Deep Socket and Ratchet Driver

1. Begin by removing the head board from the bed.
2. Using the parts provided, place the mount on the floor and add the hardware to the large tube weldment. See below for example.



***NOTICE:*** Be sure to put the 4 bolts through and make sure the split ring lock washer is between the nut and the white two hole plate.

3. Next place the tube onto the bed head section by using the tabs on the tube. The tabs should be placed in the slot on the bracket welded to the bed. Please see below for example.



4. Tighten the 3/8-16 nut and bolt until firmly tight; use a 9/16 wrench and 9/16 deep socket ratchet.


**NOTICE: BE SURE THE TWO HOLE PLATE IS FLUSH WITH THE BRACKET ON THE BED.**




5. Once the tube has been firmly fastened to the bed frame it is time to add the long extending tube to the bed. The end of the tube that has no plug should be slid over the 1 ¼ solid steel rod.

6. It is now time to assemble the extending tube. Locate each tube from the container and hardware (3/8-16 x 3 inch hex head bolts – Qty 2). Take one bolt and slide into the bushings. Once through, add the 3/8-16 nylon nut. Tighten using the 9/16 wrenches. Repeat on the other side.



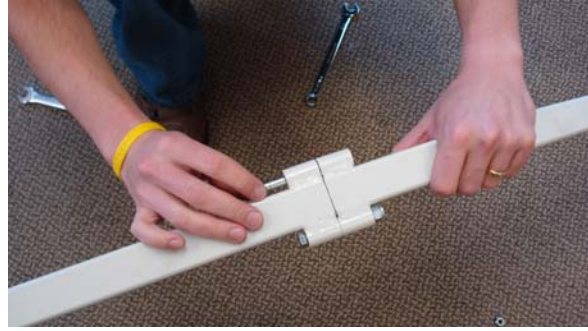

**NOTICE: BE SURE THAT THE TRAPEZE IS SEATED COMPLETELY AND THE BLACK BOLT HEAD CAN BE SEEN.** See below for example.

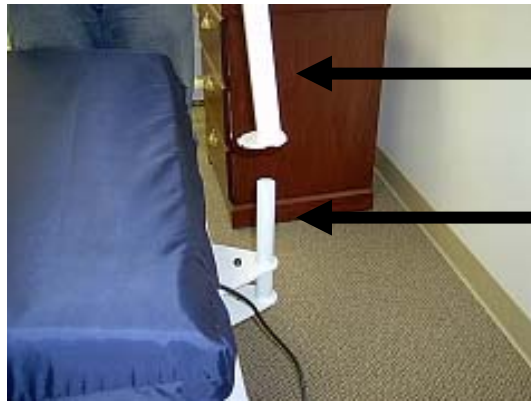

**Extending Tube**

**1 1/4 Solid Steel Rod**

7. Now it is time to assemble the handle to the extending tube. With one hand take the handle and with the other hand slide the metal chain through the zinc plated bracket. Once the desired height is achieved, turn the chain so it seats properly into the bracket.

**NOTICE: THE CHAIN MUST BE COMPLETELY INTO THE GROOVE OF THE BRACKET BEFORE USE.** See below for example.







## **OPERATING INSTRUCTIONS**

---

The trapeze has 5 distinct positions. It can either be directly over the bed, positioned against the wall on either side and/or in between those positions on both sides. These positions can be achieved by the following steps:

1. Using two hands, slide by lifting the extending tube up slightly until the bolt head no longer prevents the tube from moving side to side.



2. Now turn the tube to the desired position and then slowly lower the tube until it is in one of the five distinct positions.

***NOTICE: IT MUST BE IN ONE OF THE FIVE LOCATIONS; OTHERWISE THE TRAPEZE IS NOT LOCKED. See below for example.***



3. It is now time to adjust the horizontal reach of the trapeze. To adjust, raise the zinc plated plate towards the ceiling until it stops and then slide either forward or backwards until desired position is achieved. See below for example.

