


Resmed S7 H2i Instructions: Humidification Bipap chamber (H2i)




1. To remove humidification chamber, place thumb on top of chamber and grab lever on the bottom of the front of humidifier. Gently pull away from the machine.



2. Unlatch each side of humidifier.

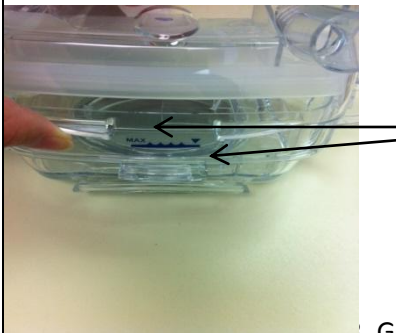

3. Gently pull apart chamber by pulling the front two plastic pieces apart from each other.

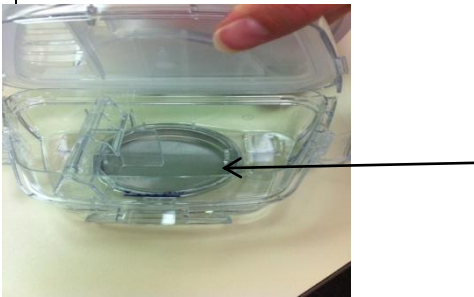

4. Fill bottom portion with distilled water BELOW the maximum line.

5. Attach the top portion of humidifier, making sure the bottom portion interlocks with the seal.

6. Latch each side and gently place back into front of Bipap humidifier until it clicks.

****Never move Bipap with water in the humidifier.**